

# HEART CONNECT™ Quick Start Guide



## RhythmCARE is your online support



Call RhythmCARE at 1800 RMCARE (762273)

Monday - Friday 9:00 - 5:00pm AEST


Excl National public holidays

After Business Hours, call 1800 245 559

Heart Connect is available for:

- ✓ Device FU
- ✓ Troubleshooting
- ✓ S-ICD Screening
- ✓ MRI activation / deactivation

## Connect the programmer to the internet

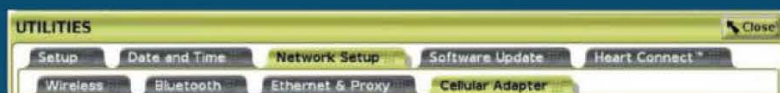

Ethernet & Proxy connects to a local area network

Wireless connects to public/open Wi-Fi networks, WPA and WPA2

Cellular Adapter connects to a cellular network (4G)

CAUTION: The law restricts these devices to sale by or on the order of a physician. Indications, contraindications, warnings, and instructions for use can be found in the product labelling supplied with each device or at [www.IFU-BSCI.com](http://www.IFU-BSCI.com). Products shown for INFORMATION purposes only and may not be approved or for sale in certain countries. This material is not intended for use in France.

2023 Copyright © Boston Scientific Corporation or its affiliates. All rights reserved.

## Real-time support with Heart Connect


Scan to learn how to make a Heart Connect call



**Boston Scientific**  
Advancing science for life™

## Start an online meeting with the RhythmCARE team

- ① Call RhythmCare at 1800 RMCARE (762273) to establish a solid Audio Connection
- ② Interrogate the patient's implanted device
- ③ Go to the Utilities Menu Heart Connect
- ④ Press the Meet Now button from the Heart Connect main
- ⑤ Press the Call button to start the meeting (DO NOT select a contact)
- ⑥ Read the Terms of Service and press "I Agree" to continue
- ⑦ Provide the Meeting ID number to allow the RhythmCARE Specialist to join call

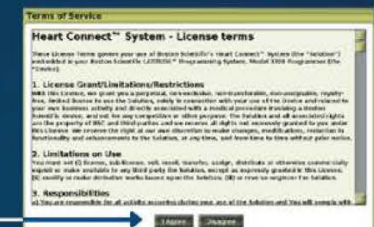

Boston Scientific Pty Ltd | PO Box 332 Botany NSW 1455 Australia | Tel +61 2 8063 8240

8100

CRM-1761104-AC