



LATITUDE™ NXT How to download a report

How to Download a Boston Scientific Pacemaker, Defibrillator or S/ICD Report

1 Login to the Boston Scientific LATITUDE NXT website

2 Click on **Search Patients**

3 Enter **Patients name**

4 Click **Search**

Login to the Boston Scientific LATITUDE NXT website

<https://www.latitude.bostonscientific.com/>

5 Click **Reports Menu**

Patient ID/ Patient/ Device	Review Status	Latest Device Transmission	Review Reason	Next Remote Follow-up	Monitoring Status/ Date	Actions
32101221 Pett, Phillip ACCOLADE L301	Viewed	31 May 2025	Implanted Device Alert	10 Aug 2025	Communicator Not Connecting 16 Jun 2025	Reports Menu

5 Click **Reports Menu**


6 Select **Combined Follow up** (Pacemaker/Defibrillator) or **Quick Notes Report / S-ICD Summary Report** (S/ICD)

7 Click **Generate Report(s)**

Select **Combined Follow up** (Pacemaker/Defibrillator) or **Quick Notes Report / S-ICD Summary Report** (S/ICD)

for online support,
scan QR or call

RhythmCARE 1800 528 488

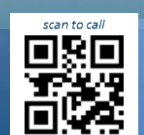


How to Download a Boston Scientific Loop (ICM) Report

1 Login to the Boston Scientific **LATITUDE** website

<https://www.latitude.bostonscientific.com/>

2 On the top right, Click **Switch to LATITUDE Clarity**

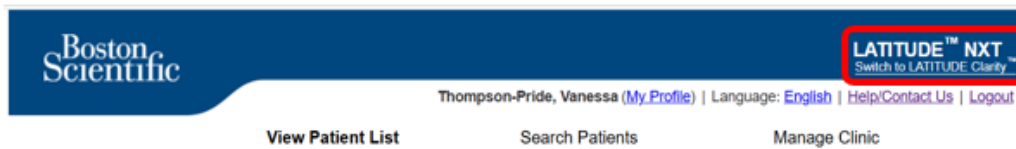

View Patient List

For Review

Patient Info	Review Reason ↓	Reason For Monitoring	Review Status	Latest Device Transmission	Next Follow-up	Patient Notes	Action
<input type="checkbox"/> Thomas, William <small>ID: Demo-9411533 DOB: 12 Jan 1967</small>	<input type="checkbox"/> Pause: 8.7 s <small>5 Event(s)</small>	<input checked="" type="checkbox"/> Suspected AF	<input type="checkbox"/> Viewed	01 Jun 2025	03 Jul 2025	M312 LUX-Ox II+	<input checked="" type="checkbox"/>

3 In the Search bar, search for your Patient by Name

4 Click on the PDF Icon

Reports Menu for Demo-9411533 (Thomas, William)

Reports

- Follow-up Report**
Includes alerts, Presenting S-ECG, event list, Counters and Settings and AF Overview.
 - Event list timeframe:
 - Include Trends and Histograms
- Event Detail Report
Includes details for each event manually selected from the Event Logbook.
- Most Recent Presenting S-ECG Report
- Programming Report

5 Select **Follow-up Report**

GENERATE REPORT(S) CANCEL

6 Click **Generate Report(s)**

for online support,
scan QR or call

RhythmCARE 1800 528 488

