



Patient Enrolment Guide

LATITUDE™ NXT



Patient Enrolment Guide

LATITUDE™ NXT

Boston
Scientific
Advancing science for life™

In this guide you will be shown the steps to successfully enrol a new patient into LATITUDE NXT™



<https://www.latitude.bostonscientific.com/>



1 Open browser and search **latitude.bostonscientific.com**



2 From the dropdowns select **Australia** then **English**

3 Click **LATITUDE NXT**

Patient Enrolment Guide

LATITUDE™ NXT

Boston Scientific LATITUDE™ NXT / LATITUDE Clarity™

LATITUDE™ NXT / LATITUDE Clarity™

Please enter your User ID and Password to enter the Boston Scientific LATITUDE Clinician website

*Indicates Required Field

*User ID:

*Password:

[Change Country/Language](#): United Kingdom/English

[Forgot Password?](#)

For additional assistance, call [LATITUDE™ Customer Support](#)

©2023 Boston Scientific Corporation or its affiliates. All rights reserved. C€2797
MD Unique Device Identifier (01) 0 0802526 12345 6 (8012) R6.1.5

Boston Scientific LATITUDE™ NXT
Switch to LATITUDE Clarity™

De Leon , Allan ([My Profile](#)) | Language: [English](#) | [Help/Contact Us](#) | [Logout](#)

View Patient List Search Patients **Manage Clinic**

Manage EMR Integration Manage Clinic Users Manage Clinic Settings

View Patient List

Viewing Patient Group: [All Patient Groups](#)

[All Patients](#) (11) | [For Review](#) (2) | [Missed Follow-up](#) (3) | [Not Monitored](#) (11) | [Not Scheduled](#) (0)

[Reports Menu for Selected Patients](#) [Dismiss From Review List](#) [Print Patient List](#) [Show More](#)

Patient ID/ Patient/ Device	Review Status	Latest Device Transmission	Alerts	Review Reason	Next Remote Follow-up	Monitoring Status/ Date	Actions
AR2016 <input type="checkbox"/> Smith, Peter EMBLEM MRI S-ICD A219	Viewed	26 May 2025		Implanted Device Alert	01 Sep 2025	Communicator Not Connecting 16 Jun 2025	<input checked="" type="checkbox"/> Dismiss From Review List <input type="button" value="Reports Menu"/>
32101221 <input type="checkbox"/> Petit, Philippe ACCOLADE L301	Viewed	31 May 2025		Implanted Device Alert	10 Aug 2025	Communicator Not Connecting 16 Jun 2025	<input checked="" type="checkbox"/> Dismiss From Review List <input type="button" value="Reports Menu"/>

View Patient List 1 - 2 of 2 [Show More](#)

©2023 Boston Scientific Corporation or its affiliates. All rights reserved.

3 Enter **User ID** and **Password**
Click **Login**

4 Select **Manage Clinic**
then **Enrol Patient**

Patient Enrolment Guide

LATITUDE™ NXT

Boston Scientific LATITUDE™ NXT
Switch to LATITUDE Clarity™

De Leon, Allan (My Profile) | Language: English | Help/Contact Us | Logout

View Patient List Search Patients Manage Clinic

Manage EMR Integration Manage Clinic Users Manage Clinic Settings **Enroll Patient**

Enroll Patient Step 1 of 3: Enter implanted device and patient's date of birth.

*Indicates Required Field

Implanted Device Information

Device Model: 1
Device Serial Number: 2
Patient Date of Birth: (e.g.: 24/01/1950) 3

Continue

Is this a Replacement Device?
To update device information or enter a new serial number, locate an existing Patient via the [Patient List](#).

©2023 Boston Scientific Corporation or its affiliates. All rights reserved.

Boston Scientific LATITUDE™ NXT
Switch to LATITUDE Clarity™

De Leon, Allan (My Profile) | Language: English | Help/Contact Us | Logout

View Patient List Search Patients Manage Clinic

Manage EMR Integration Manage Clinic Users Manage Clinic Settings **Enroll Patient**

Enroll Patient Step 2 of 3: Enter/Edit Patient Information

*Indicates Required Field

General Information

Patient Information

Last Name: Phone Number 1: (country code + number, e.g. 44 123 456 789)
First Name: Phone Number 2: (country code + number, e.g. 44 123 456 789)
Middle Name: Date of Birth: (e.g.: 24/01/1950)
Suffix: Search Tags:
Patient ID: Patient Time Zone: (GMT+00:00) Western Time - London

Patient Group Membership

Primary Patient Group: Secondary Patient Group:
-No Patient Group Selected- -No Patient Group Selected-

- From the dropdown select **Device Model**
Enter **Serial Number**
Enter **Patient Date of Birth**
Click **Continue**

- Enter Patient Information
Phone Number 1 = **Mobile Number**
Phone Number 2 = **Landline** (include area code)

Patient Enrolment Guide

LATITUDE™ NXT

Equipment

Communicator

Model: i

Serial Number: i

Blood Pressure Monitor

Model:

Serial Number: i

Weight Scale

Model:

Serial Number: i

Clinician Information

Referring Clinician

Last Name:

E-mail Address:

First Name:

Phone Number: (country code + number, e.g. 44 123 456 789)

Patient Consent

I confirm that the patient has signed Boston Scientific Corporation's LATITUDE™ data authorization form as per the model communicated by Boston Scientific to the clinic.

- 7 Select Communicator **Model** from dropdown
Enter Communicator **Serial Number**

Confirm **Patient** has signed consent
Click **Enrol Patient**

Boston Scientific LATITUDE™ NXT
Switch to LATITUDE Clarity™

De Leon, Allan (My Profile) | Language: English | Help/Contact Us | Logout

View Patient List Search Patients Manage Clinic

Manage EMR Integration Manage Clinic Users Manage Clinic Settings **Enroll Patient**

Enroll Patient Step 2 of 3: Enter/Edit Patient Information

The Communicator is not allocated to your clinic. If the model and serial number entered are correct press "Enroll Patient" at the bottom of the page.

For additional assistance, contact **LATITUDE™ Customer Support** i.

Indicates Required Field

- 8 A pop-up message will appear to ensure details are correct

Click **Enrol** at bottom of screen

Patient Enrolment Guide

LATITUDE™ NXT

9

A Confirmation Page will advise of a **Successful Enrolment**

Boston Scientific LATITUDE™ NXT
Search to LATITUDE Clinic™

De Leon , Allan (My Profile) | Language: English | [Help/Contact Us](#) | [Logout](#)

[View Patient List](#) [Search Patients](#) [Manage Clinic](#)

[Manage EMR Integration](#) [Manage Clinic Users](#) [Manage Clinic Settings](#) **[Enroll Patient](#)**

Enroll Patient Step 3 of 3: Print Confirmation Page

Enrolled on: 08 Aug 2025
Enrolled by: De Leon , Allan

Enrollment Complete

✔ You have successfully enrolled this patient. [Enroll Another Patient](#)

Confirmation Details [Print Confirmation Page](#)

General Information

Patient Information

Last Name	Smith	Phone Number 1	0412 123 123
First Name	John	Phone Number 2	07 1234 1234
Middle Name		Patient ID	John Smith
Suffix		Date of Birth	14/03/1980
		Search Tags	
		Patient Time Zone	(GMT+00:00) Western Time - London

Implanted Device Information	Patient Group Membership		
Model	L200 PROPONENT	Primary Patient Group	LUX-Dx ICM Group
Serial Number	000002	Secondary Patient Group	

scan to call



for online support, scan QR or call

RhythmCARE

1800 528 488