



LATITUDE™ NXT Transfer Patient

In this guide you will be shown the steps to successfully transfer an existing patient into your LATITUDE NXT™ Clinic

- 1 Open browser and search **latitude.bostonscientific.com**

<https://www.latitude.bostonscientific.com/>



LOGIN

- 2 From the dropdowns select **Australia**, then **English**
Click **LATITUDE NXT**
- 3 Enter **User ID** and **Password**. Click **Login**

MANAGE CLINIC

- 4 Select **Manage Clinic** then **Enrol Patient**
- 5 From the dropdown select **Device Model**
Enter **Serial Number**
Enter **Patient Date of Birth**

Click **Continue**

- 6
 - Enter **Patient ID** (16 Character limit)
 - For patients transferring *from* another clinic, **click checkbox** confirming your clinic is now following the patient.
 - Select **Primary Patient Group** from dropdown

ENROL

- 7 Confirm **Patient has signed consent**
Click **Enrol Patient**

- 8 A Confirmation Page will advise of a **Successful Transfer and Enrolment of Patient** into your clinic

for online support,
scan QR or call

RhythmCARE 1800 528 488

