




LATITUDE™ NXT Updating Patient Phone Numbers

In this guide you will be shown the steps to successfully update patient phone numbers into LATITUDE NXT™

- 1 Open browser and search **latitude.bostonscientific.com**

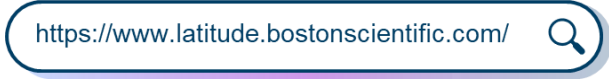


LOGIN

- 2 From the dropdowns select **Australia**, then **English**
Click **LATITUDE NXT**
- 3 Enter **User ID** and **Password**. Click **Login**


SEARCH

- 4 Select **Search Patients**
- 5 Type in **Last Name** and **First Name**, then Click **Search**
- 6 Click on the Patient's name

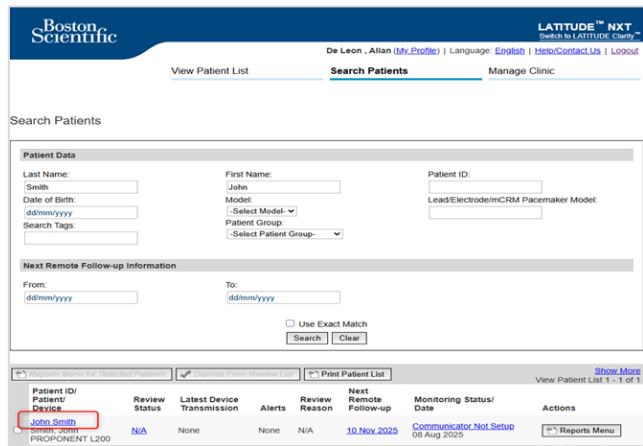

UPDATE

- 7 Click **Edit/View Patient and Equipment Information**
- 8 Update Phone Numbers (mandatory fields)
Phone Number 1 = Mobile
Phone Number 2 = Landline (include area code)
- 9 Click **Save** at the bottom of the screen

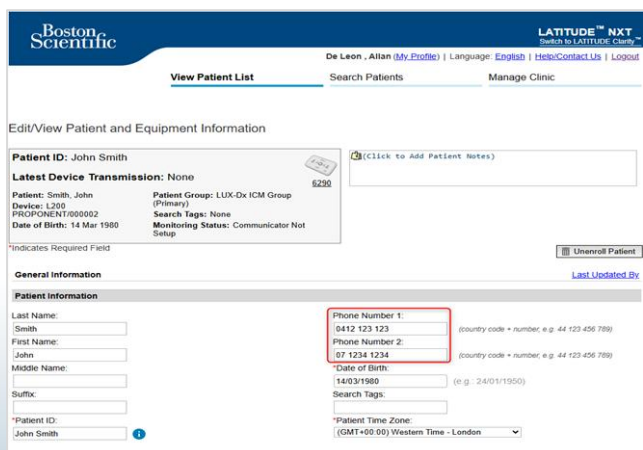

for online support,
scan QR or call

RhythmCARE 1800 528 488

